

La-Z-Boy
Lift and Rise
Power Recliner
Operation and Care
User Guide

For more information visit
www.la-z-boy.co.uk

La-Z-boy[®]

Version 0126

Contents

1 Setup

Intended use

Unpacking and Assembly

Transformer Connection

2 Operating instructions

Safety

Limitations of Use

Handset Control




Intended Use

- This chair is intended to help any adult (with a maximum weight limit of 110kg) who requires the luxury of a fully reclining, upholstered chair with a lifting function. The lifting function helps the user to their feet when leaving the chair and to a sitting position when getting into the chair.
- The chair is intended to be used in a domestic environment. If this chair has been purchased by a nursing home or other public or commercial enterprise, please establish whether electrical and other classifications are being met. Contact the seller of this chair or the manufacturer if this is not possible.
- The chair is designed, as much as practically possible, to conform to the natural movement of an average person. It is not a medical device and professional advice must be taken if certain movements might aggravate a medical condition.

Unpacking and Assembly

- There should be two boxes with your La-Z-Boy chair. The large box contains the seat/base frame and the small box contains the back. Care must be taken on opening the boxes as fabrics can be damaged with sharp objects.
- There are two metal sleeves on the outside of the chair back which must be located on the two back hinges.
- The back hinges protrude above the seat near the back of the chair. The back should be pushed onto the hinges by standing behind the chair. Locate the brackets onto both pivot points and push downwards firmly. The back is then locked into position.

Transformer Connection

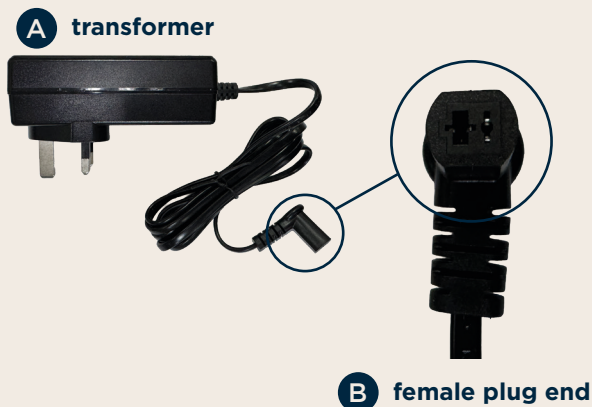

- Before plugging the transformer **(A)** in to the wall socket, connect the female plug end **(B)** into the socket at the rear of the chair arm.
- **Plug the 3-pin plug** into the 13A, 230V mains power on your wall.
- **Ensure** the La-Z-Boy chairs at least 50cm away from the wall.
- **Turn the power on.** Before using your La-Z-Boy chair, please check that there is enough clearance from the wall by lowering the back fully.

Safety

- The La-Z-Boy chair must be clear of any obstructions before use.
- This chair is designed for use by one adult at a time only.
- There are moving parts, which are being driven by powerful actuators (motors) and care should be taken at all times.
- **Do not** let children operate or play near the chair.
- The footrest is 'sprung' to reduce the risk of entrapment when closing, but care should be taken when the footrest is being lowered and extended/raised.
- **Ensure children and pets** are not in the chair's vicinity when any of the motorised functions of the chair are being utilised.
- **Do not** sit on the footrest when it is raised.
- **Do not** sit on the back when it is lowered.
- If used incorrectly, the chair may become unstable and dangerous.
- **NOTE:** This chair is heavy and requires two people to lift.

Limitations of Use

- **Do not** sit on the footrest when it is raised.
- **Do not** allow anyone to sit on the footrest when the user is in the chair.

Handset Control

1 BACK LOWER

When this button is pushed, the back cushion will lower to the reclined position.

2 BACK RAISE

When this button is pushed, the back cushion will rise to the upright position.

3 FOOTREST RAISE

When this button is pushed, the footrest will rise to the elevated position.

4 LOWER FOOTREST THEN LIFT

When this button is pushed, the footrest will lower to the downward position. Once the footrest is fully lowered, the chair will then start to tilt and lift the user.

5 FOOTREST RAISE AND BACK LOWER

When this button is pushed, the footrest will rise to the elevated position and the back will lower to the reclined position.

6 LOWER FOOTREST, RAISE BACK AND THEN LIFT

When this button is pushed, the footrest will lower to the downward position and the back will rise to the upright position. Once the footrest and back are in position, the chair will then start to tilt and lift the user. During the lift and tilt movement, the back will automatically adjust to help the user maintain an upright posture.

Never let children play with the handset.

If the chair is likely to be unattended for any period of time, and children or people requiring special supervision have access to this chair, the transformer must be removed from the main power supply and stored in a secure place.

NOTE: Your Lift and Rise chair requires 2 x 9V batteries to enable the chair to open/close in case of a powercut/shortage etc. The battery backup allows 1 x open/close and/or lift and rise function.



1

2

3

4

5

6

Further product documentation
can be found at
la-z-boy.co.uk/productguides
or scan the QR code.



La-Z-boy[®]

www.la-z-boy.co.uk